

# FC20 Exam

*Lifeloc Technologies, Inc.*

Mark your answers clearly by CIRCLING or highlighting the letter (A, B, C, or True, False etc.) that corresponds with your selected answer.



Example:

Which manufacturer treats their customers the best?

- A. Drunks-are-us
- B. **Lifeloc Technologies**
- C. Ted's Testers. Inc

**PLEASE PRINT CLEARLY**

Student Name: \_\_\_\_\_

Agency Name: \_\_\_\_\_

Address: \_\_\_\_\_

City, State & Zip: \_\_\_\_\_

E-Mail: \_\_\_\_\_

Date: \_\_\_\_\_

Lifeloc Model: \_\_\_\_\_

Return this completed form to:  
**mastertrainer@lifeloc.com**

Lifeloc Technologies  
12441 West 49<sup>th</sup> Ave., Unit 4  
Wheat Ridge, CO 80033

## FC Exam

1. **What type of batteries does the FC require?**
  - A. Standard AAA only
  - B. Rechargeable only
  - C. Standard AA only
  - D. Either Standard AA or NiMH Rechargeable
  
2. **When does the “Precise Volume” trigger mode take the breath sample?**
  - A. At the end of breath
  - B. After 2 seconds
  - C. After 1.5 liters of breath
  
3. **What button works as the “home” key?**
  - A. Execute
  - B. Power
  - C. Function
  
4. **The FC20 is designed to operate in which internal temperature range?**
  - A. 0 - 23°C
  - B. 7 - 35°C
  - C. 0- 55°C
  
5. **Conducting a Manual Override of an Auto Test requires that you push and release the Execute button to take a breath sample?**
  - A. True
  - B. False
  
6. **You should instruct the subject to blow as hard as they can into the FC.**
  - A. True
  - B. False
  
7. **The FC20 automatically compensates for which event?**
  - A. Daylight Savings Time
  - B. Leap Year
  - C. Time Zones
  
8. **You can manually override an Auto Test.**
  - A. True
  - B. False
  
9. **What substance will destroy the fuel cell sensor in the FC?**
  - A. Mouthwash
  - B. Nicotine
  - C. Gum
  
10. **Passive Testing will provide a numeric BAC reading.**
  - A. True
  - B. False

Return this completed form to:  
**mastertrainer@lifeloc.com**

Lifeloc Technologies  
12441 West 49<sup>th</sup> Ave., Unit 4  
Wheat Ridge, CO 80033

FC Exam

- 11. It is important to keep the fuel cell moist.**  
A. True  
B. False
- 12. How many tests does the FC20 hold in memory?**  
A. 4,000  
B. None  
C. 250
- 13. What button works as the “save” key?**  
A. Power  
B. Execute  
C. Function
- 14. A new mouthpiece should be attached before each direct test.**  
A. True  
B. False
- 15. The FC has an adjustable backlight display.**  
A. True  
B. False
- 16. How many digits are displayed in a direct test BAC result?**  
A. 2  
B. 3  
C. 4
- 17. What does the User Prompt *Flow Error* mean?**  
A. There is not alcohol detected.  
B. The breath flow ended before the subject blew them minimum liters of breath.  
C. External interference has been detected.
- 18. What should you do if the User Prompt *Pump Reset Needed* is displayed?**  
A. Replace the batteries.  
B. Have the subject blow again.  
C. Follow the onscreen instructions.

Please send completed test to [mastertrainer@lifeloc.com](mailto:mastertrainer@lifeloc.com) to receive your certificate of completion. Certificate processing takes 7-10 business days.

Return this completed form to:  
**mastertrainer@lifeloc.com**

Lifeloc Technologies  
12441 West 49<sup>th</sup> Ave., Unit 4  
Wheat Ridge, CO 80033